

XBACK FITTING & CARE INSTRUCTIONS

XBACK LS MODEL T103

FOLLOW THESE INSTRUCTIONS FOR WEARING AND CARING FOR YOUR XBACK BRACE

Steps for putting on and adjusting your brace:



Remove the LS from the package.



Open the brace.



Loosen the 2 compression straps and temporarily attach to the small squares on the side of the brace.



Center the posterior panel on the back.



Wrap the left panel around the abdomen.



Wrap right panel around abdomen and secure to left panel.



Grasp the compression straps with the thumbs and loosen from small squares.



Pull both forward away from the body simultaneously until the desired compression is achieved.



Secure the compression straps to front of the brace.



Properly Fitted Brace

Care Instructions:

Hand wash only in cool water with a mild liquid soap and lay flat to air dry. When drying do not expose to direct heat or sunlight. Do not dry clean. If your brace has a rigid shell, the shell may be cleaned with a mild topical cleaner. Do not use strong detergents or bleaches as both may damage the brace materials.

Warranty:

All XBACK braces are warranted to be free of material or workmanship defects for 5 years. Any defective product will be repaired or replaced at the discretion of XBack Bracing Services, Inc. after review of the defect by Xback Bracing Services, Inc. There are no other implied or expressed warranties.



XBACK®

Simply Innovative Braces™

XBack Bracing Services, Inc.
P.O. Box 100
341A West Main Street
Birdsboro, PA 19508 USA
610.404.4900 Phone
610.404.4905 Fax
info@xbackbrace.com
www.xbackbrace.com

