


# XBACK FITTING & CARE INSTRUCTIONS

## PROLIFT-PLUS ADJUSTABLE MODEL T109

FOLLOW THESE INSTRUCTIONS FOR WEARING AND CARING FOR YOUR XBACK BRACE


### Steps for putting on and adjusting your brace:

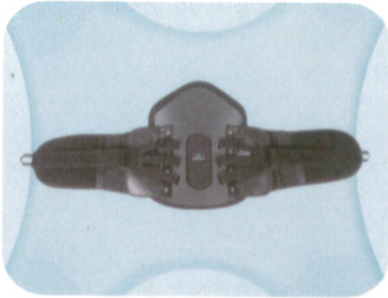

1. Open brace and lay face down on a flat surface. Loosen compression straps.



2. Open alligator clamp adjustable panel.

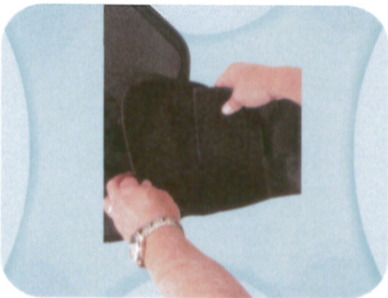

3. Adjust for exact patient size.



4. Close alligator clamp panel.



5. Center the posterior panel in the middle of your patient's back.



6. Close the front panel low and snug on the patient's abdomen.



7. Tighten the outside compression straps to the desired level of support, and anchor on the front abdominal panel.

### Care Instructions:

Hand wash only in cool water with a mild liquid soap and lay flat to air dry. When drying do not expose to direct heat or sunlight. Do not dry clean. If your brace has a rigid shell, the shell may be cleaned with a mild topical cleaner. Do not use strong detergents or bleaches as both may damage the brace materials.

### Warranty:

All XBACK braces are warranted to be free of material or workmanship defects for 5 years. Any defective product will be repaired or replaced at the discretion of XBack Bracing Services, Inc. after review of the defect by Xback Bracing Services, Inc. There are no other implied or expressed warranties.



**XBACK**<sup>®</sup>

Simply Innovative Braces™

XBack Bracing Services, Inc.  
P.O. Box 100  
341A West Main Street  
Birdsboro, PA 19508 USA  
610.404.4900 Phone  
610.404.4905 Fax  
info@xbackbrace.com  
www.xbackbrace.com

