


XBACK FITTING & CARE INSTRUCTIONS

PROLIFT-PLUS MODEL T109

FOLLOW THESE INSTRUCTIONS FOR WEARING AND CARING FOR YOUR XBACK BRACE

Steps for putting on and adjusting your brace:

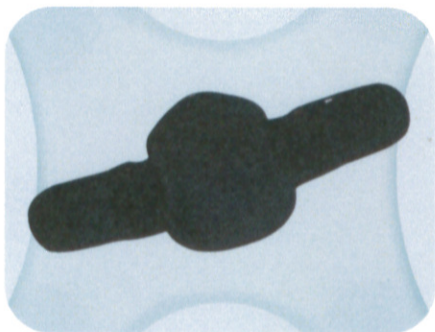

1. Open the XBack ProLift-Plus back brace.



2. Center the posterior panel in the middle of your patient's back.



3. Close the front panel low and snug on the patient's abdomen.



4. Tighten the outside compression straps to the desired level of support, and anchor on the front abdominal panel.

Care Instructions:

Hand wash only in cool water with a mild liquid soap and lay flat to air dry. When drying do not expose to direct heat or sunlight. Do not dry clean. If your brace has a rigid shell, the shell may be cleaned with a mild topical cleaner. Do not use strong detergents or bleaches as both may damage the brace materials.

Warranty:

All XBACK braces are warranted to be free of material or workmanship defects for 5 years. Any defective product will be repaired or replaced at the discretion of XBack Bracing Services, Inc. after review of the defect by Xback Bracing Services, Inc. There are no other implied or expressed warranties.



XBACK®

Simply Innovative Braces™

XBack Bracing Services, Inc.
P.O. Box 100

341A West Main Street
Birdsboro, PA 19508 USA
610.404.4900 Phone
610.404.4905 Fax
info@xbackbrace.com
www.xbackbrace.com

