

# XBACK FITTING & CARE INSTRUCTIONS

## XBACK LS MODEL T103 Adjustable

FOLLOW THESE INSTRUCTIONS FOR WEARING AND CARING FOR YOUR XBACK BRACE

### Steps for putting on and adjusting your brace:



Open the brace.



Loosen the 2 compression straps and temporarily attach to the small squares on the side of the brace.



Open alligator clamp adjustable panel.



Adjust alligator clamp panel for exact patient size.



Close alligator clamp panel.



Center the posterior panel on the back.



Wrap the right panel around the abdomen and secure to left panel.



Pull both forward away from the body simultaneously until the desired compression is achieved.



Secure the compression straps to front of the brace.



Properly Fitted Brace

### Care Instructions:

Hand wash only in cool water with a mild liquid soap and lay flat to air dry. When drying do not expose to direct heat or sunlight. Do not dry clean. If your brace has a rigid shell, the shell may be cleaned with a mild topical cleaner. Do not use strong detergents or bleaches as both may damage the brace materials.

### Warranty:

All XBACK braces are warranted to be free of material or workmanship defects for 5 years. Any defective product will be repaired or replaced at the discretion of XBack Bracing Services, Inc. after review of the defect by Xback Bracing Services, Inc. There are no other implied or expressed warranties.



# XBACK®

Simply Innovative Braces™

XBack Bracing Services, Inc.  
P.O. Box 100  
341A West Main Street  
Birdsboro, PA 19508 USA  
610.404.4900 Phone  
610.404.4905 Fax  
info@xbackbrace.com  
www.xbackbrace.com

