

XBack EDGE LS

The XBack Edge LS LSO brace provides coverage and support from the sacrum through the L1 vertebrae. Hydrostatic lift helps take the weight of the abdomen off the lower spine which helps decompress the vertebrae providing consistent relief for low back pain.



Manufactured by:

XBack Bracing Services
PO Box 100
Birdsboro, PA 19508

SIZING CHART

SMALL	23-29 inches
MEDIUM	29-35 inches
LARGE	35-42 inches
X-LARGE	42-50 inches
XX-Large	50-58 inches
XXX-Large	58-66 inches

**Brace sizing ranges are only guidelines. Care should be taken when choosing the appropriate size brace to ensure proper fitting.

Care Instructions

- ◇ Hand wash only
- ◇ Clean plastic panels with mild topical cleaner
- ◇ Hand wash the padding and straps with a mild detergent

XBack EDGE LS Approved HCPC Code:

L0642 or L0627 (L0627 Depends upon if HCP adjusts ie-customized to fit a specific patient by an individual with expertise.)



XBack EDGE LS Model T49

COMMON INDICATIONS

- ◇ Bulging Discs
- ◇ Chronic Back Pain
- ◇ Degenerative Disc Disease
- ◇ Herniated Discs
- ◇ Osteo arthritis/ Arthritis
- ◇ Scoliosis
- ◇ Spinal Stenosis
- ◇ Spondylolysis
- ◇ Spondylolisthesis
- ◇ Post-Op Stabilization
- ◇ Kyphosis pain
- ◇ Mechanical lower and mid-back pain

EDGE LS LSO



FOLLOW THESE

Step-by-Step

Fitting Instructions




1. Stretch brace out fully across patient's back & align brace in center of back.



2. Overlap front panels of brace and secure Velcro. Brace should be snug but comfortable.



3. Pull bilateral tension straps until desired level of support is achieved.

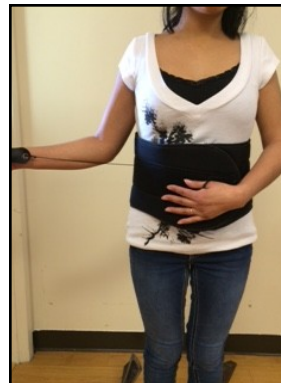

4. Secure tension straps with Velcro side facing down.



5. Check with patient to ensure that fully fitted LSO is secure and comfortable